

**510(k) SUBSTANTIAL EQUIVALENCE DETERMINATION
DECISION SUMMARY
ASSAY ONLY TEMPLATE**

A. 510(k) Number:
k062839

B. Purpose for Submission:

To add additional organism groups to the antibiotics tetracycline, trimethoprim/sulfamethoxazole, and vancomycin on the Sensititre® *Haemophilus influenzae* / *Streptococcus pneumoniae* (HP) MIC and Sensititre® Susceptibility Plates

C. Measurand:	Range µg/mL
Tetracycline	0.5-8
Trimethoprim/sulphamethoxazole	0.5/9.5-4/76
Vancomycin	0.5-4

D. Type of Test:
Quantitative Antimicrobial Susceptibility Test (AST) growth based fluorescence

E. Applicant:
TREK Diagnostic Systems, Inc.

F. Proprietary and Established Names:
Sensititre® *Haemophilus influenzae* / *Streptococcus pneumoniae* (HP) MIC plates

G. Regulatory Information:

1. Regulation section:
866.1640 Antimicrobial Susceptibility Test Powder
2. Classification:
II
3. Product code:
JWY-manual readings of AST testing of >16 hour incubation
LRG Automated readings of AST of >16 hour incubation.
4. Panel:
83 Microbiology

H. Intended Use:

1. Intended use(s):

The Sensititre® *Haemophilus influenzae* / *Streptococcus pneumoniae* (HP) MIC Susceptibility plate is an *in vitro* diagnostic product for clinical susceptibility testing of *Haemophilus influenzae*, *Streptococcus pneumoniae* and *Streptococcus* species.

2. Indication(s) for use:

This application is for the addition of *Streptococcus* species (spp.) to tetracycline (0.5-8 µg/mL), trimethoprim/sulphamethoxazole (0.5/9.5 – 4/76 µg/mL), and vancomycin (0.5-4 µg/mL) for use with the Sensititre® *Haemophilus influenzae*/*Streptococcus pneumoniae* (HP) MIC Susceptibility Plates.

3. Special conditions for use statement(s):

For prescription use only

4. Special instrument requirements:

Automated readings are performed on the Sensititre® AutoReader or Sensititre® ARIS®.

I. Device Description:

Sensititre® MIC Susceptibility plate MIC panels are multi-well plastic microtitre plates, precision dosed with dried, stabilized antimicrobics. This is similar to the micro broth dilution methods and can provide both qualitative and quantitative susceptibility results. The medium required for testing *Streptococcus* is Sensititre® Mueller-Hinton (MH) broth with 2 – 5% lysed horse blood with a final organism density of 5×10^5 colony units (CFU/mL). This is then incubated in a non CO₂ incubator for 20 – 24 hour and read manually for growth or using the Sensititre® autoread.

J. Substantial Equivalence Information:

1. Predicate device name(s):

Pasco MIC and MIC/ID Panels

2. Predicate 510(k) number(s):

K033119

3. Comparison with predicate:

Similarities		
Item	Device	Predicate
Intended use	an <i>in vitro</i> diagnostic product for clinical susceptibility testing of gram negative and gram positive organisms.	Same
Inoculum	Prepared from colonies using the direct inoculation method	Same
Growth medium	Mueller – Hinton (MH) broth with 2 – 5% lysed horse blood	Same
Inoculation method	Direct equated to a 0.5 McFarland	Same
Differences		
Item	Device	Predicate
Type panel	Dried antibiotics	100 µl/well frozen
Incubation	20 - 24 hours	16-24 hours
Technology	Fluorescence detection of growth	Turbidity detection of growth
Reading method	Visual growth and Auto read by instrumentation	Turbidity detection of growth

K. Standard/Guidance Document Referenced (if applicable):

“Class II Special Controls Guidance Document: Antimicrobial Susceptibility Test Systems; Guidance for Industry and FDA”; CLSI M7 (M100-S16) “Methods for Dilution Antimicrobial Susceptibility Tests for Bacteria That Grow Aerobically; Approved Standard.”

L. Test Principle:

The Sensititre® Autoread System utilizes fluorescence technology to read 20 -24 hour *Streptococcus pneumoniae* plates. The technology involves the detection of bacterial growth by monitoring the activity of specific surface enzymes produced by the test organism. Growth is determined by generating a fluorescent product from a non-fluorescent (fluorogenic) substrate. The non-fluorescent substrate is prepared by conjugating a fluorescent compound to the specific enzyme substrates with a bond, which prevents fluorescence. The fluorophore, is then said to be quenched. The substrate is added to the inoculum broth and dispensed into the test plates at the same time as the test organism or the plates can be prepared with the substrate already added to the plate. Enzymatic action of the bacterial surface enzymes on the specific substrates, cleave this bond releasing the fluorophore, which is now capable of fluorescence. The amount of fluorescence detected is directly related to the activity of bacterial growth. The MIC is determined by observing the lowest dilution of antimicrobial agent that inhibits growth of the organism or automatically for the detection of fluorescence.

M. Performance Characteristics (if/when applicable):

1. Analytical performance:

a. *Precision/Reproducibility:*

A reproducibility study was performed on 25 *Streptococcus* spp. These isolates were tested once for each antimicrobial at each of the three sites on the automated and manual read method demonstrating >95% reproducibility for both read methods.

b. *Linearity/assay reportable range:*

Not applicable

c. *Traceability, Stability, Expected values (controls, calibrators, or methods):*

The recommended QC isolate, *S. pneumoniae* 49619 was tested daily with acceptable results. Quality control was also performed at all sites using both manual and autoread methods. The Sensititre® results demonstrated that the system can produce QC results in the recommended range for both manual and automated read methods. The mode is the same between the reference method and the two read methods. The reference method was in range for every day tested. Table below includes the frequency of each result in the range tested.

Antimicrobial	ORGANISM	Conc ug/mL	Sensititre® Autoread	Sensititre® manual	Reference
Tetracycline	<i>S. pneumoniae</i> ATCC 49619 Exp. Range: 0.12 – 0.5 ug/ml	≤0.5	60	60	60
		1			
		2			
		4			
	<i>S. pneumoniae</i> #5 Exp. Range : ≥ 4 ug/ml	4			
		8			
		≥ 16	60	60	60
Trimethoprim/ sulfamethoxazole	<i>S. pneumoniae</i> ATCC 49619 Exp. Range: 0.12/2.4-1/19 ug/ml	≤20.5/9.5	58	60	60
		1/19	2		
		2/38			
	<i>S. pneumoniae</i> #5 Exp. Range : ≥ 2/38 ug/ml	2/38			
		4/76	18	44	39
		≥ 8/156	42	16	21
Vancomycin	<i>S. pneumoniae</i> ATCC 49619 Exp. Range: 0.12 – 0.5 ug/ml	≤0.5	60	60	60
		1			
	<i>S. pneumoniae</i> #5 Exp. Range: 0.25 – 1 ug/ml	≤ 0.5	60	60	60
		1			
		2			

Nephelometer was used at each site to standardize the inoculum and it was

calibrated each time it was switched on. A colony count from QC ATCC source was performed using direct inoculum method with acceptable results

d. *Detection limit:*
Not Applicable

e. *Analytical specificity:*
Not Applicable

f. *Assay cut-off:*
Not Applicable

2. Comparison studies:

a. *Method comparison with predicate device:*

The recommended broth dilution reference panel was prepared according to the CLSI recommendation. Clinical testing included both fresh and stock clinical isolates of *Streptococcus viridans* group and *Beta streptococcus* and a set of challenge organisms. The broth reference panel for *Streptococcus* spp. was set up on MH supplemented with 2% to 5% lysed horse blood as recommended by CLSI and incubated in a non CO2 incubator for 20 – 24 hours. The TREK plate was read both manually and using the Sensititre Autoreader between 20 and 24 hour. The comparison resulted in the following performance evaluations as reflected below. The Autoreader results and the manual results were very similar with acceptable results for both. Only one method is presented below for each antibiotic.

Summary Table for *Streptococcus* spp. other than *S. pneumoniae* with tetracycline

	EA Tot	EA N	EA %	Eval EA Tot	Eval EA N	Eval EA %	CA N	CA %	#R	min	maj	vmj
Clinical isolates	299	298	99.7	18	18	100	295	98.7	122	3	1	0
Challenge isolates	50	50	100	4	4	100	50	100	18	0	0	0
Combined	349	348	99.7	22	22	100	345	98.9	140	3	1	0

Summary Table for *Streptococcus* spp. other than *S. pneumoniae* with Trimethoprim/sulphamethoxazole

	EA Tot	EA N	EA %	Eval EA Tot	Eval EA N	Eval EA %	CA N	CA %	#R	min	maj	vmj
Clinical isolates	299	298	99.7	17	17	100	*	*	*	*	*	*
Challenge isolates	50	50	100	9	9	100	*	*	*	*	*	*
Combined	349	348	99.7	26	26	100	*	*	*	*	*	*

*Only EA can be determined for this group of organisms and Trimethoprim/sulphamethoxazole. There are no interpretive criteria.

Summary Table for *Streptococcus* spp. other than *S. pneumoniae* with vancomycin

	EA Tot	EA N	EA %	Eval EA Tot	Eval EA N	Eval EA %	CA N	CA %	#R	min	maj	vmj
Clinical isolates	299	299	100	6	6	100	299	100	0	0	0	0
Challenge isolates	50	50	100	0	0	-	50	100	0	0	0	0
Combined	349	349	100	6	6	100	349	100	0	0	0	0

EA-Essential Agreement

CA-Category Agreement

R-resistant isolates

vmj – very major discrepancies

maj - major discrepancies

min – minor discrepancies

EA is when there is agreement between the reference method and the Sensititre® panel within plus or minus one serial two-fold dilution of antibiotic. Category agreement (CA) is when the Sensititre® panel result interpretation agrees exactly with the reference panel result interpretation. Evaluable EA is when the MIC result is on scale for both the Sensititre® and the reference and have on-scale EA. The EA% is acceptable when compared to the reference method as described in the FDA guidance document, “Class II Special Controls Guidance Document: Antimicrobial Susceptibility Test (AST) Systems; Guidance for Industry and FDA”.

The charts above demonstrated that the performance for *Streptococcus* spp. other than *S. pneumoniae* was very good for both methods of reading with no visible trending observed.

The growth rate for *Streptococcus* spp. is greater than 95%.

- b. *Matrix comparison:*
Not Applicable

3. Clinical studies:

- a. *Clinical Sensitivity:*
Not Applicable

- b. *Clinical specificity:*
Not Applicable

- c. Other clinical supportive data (when a. and b. are not applicable):
Not Applicable

4. Clinical cut-off:
Not Applicable

5. Expected values/Reference range:

Antibiotic	Organism	Interpretative Criteria
Tetracycline	<i>Streptococci</i> spp. including <i>S. pneumoniae</i>	≤2 (S), 4(I), ≥8(R)
Trimethoprim/sulphamethoxazole	<i>Streptococci</i> spp.	MIC only provided
Vancomycin	<i>Streptococcus</i> spp. other than <i>S. pneumoniae</i> (Viridans group)	≤1 (S)*

* For some organism/antimicrobial agent combinations, the absence or rare occurrence of resistant strains precludes defining any results categories other than “susceptible.” For strains yielding results suggestive of a “nonsusceptible” category, organism identification and antimicrobial susceptibility test results should be confirmed. Subsequently, the isolates should be saved and submitted to a reference laboratory that will confirm results using a CLSI/NCCLS reference dilution method.

N. Proposed Labeling:

The expected value range, interpretive criteria and QC for gram positive panels are included in the package insert. The labeling is sufficient and satisfies the requirements of 21 CFR Part 809.10.

O. Conclusion:

The submitted information in this premarket notification is complete and supports a substantial equivalence decision.